

INDERMIL® flexifuze™

The latest technology in Cyanoacrylate tissue adhesive.



The Product

- IDERMIL® flexifuze™ is a sterile, liquid topical tissue adhesive.
- IDERMIL® flexifuze™ is indicated for the closure of skin wounds, including laparoscopic incisions and trauma-induced lacerations.

For the Patient

- **Flexible¹ as skin** – Acts and feels like a second layer of skin.
- **Shower proof²** – Ability to shower daily.
- **Reduced anxiety, less painful** – No need for needles, sutures or staples.
- **Excellent cosmetic result** – Less trauma to wound site, reducing scarring.
- **No discolouration** of wound site³ – Dye free formulation.
- **Microbial barrier⁴** – Reduced risk of infection.
- **Minimal exotherm⁵** – Minimal exothermic reaction.

For the Clinician

- **Flow Control Applicator⁶** – Patent protected technology, which provides a broad stroke for optimal wound coverage.
- **Cannula applicator** – Option to use the cannula applicator providing pin point accuracy.
- **Fast and easy application** – Sets in **30 - 60 seconds⁷**.
- **Clog free tip⁸** providing long working life (up to **40 minutes**) allowing for multiple use on a single patient, reducing wastage.
- IDERMIL® flexifuze™ stays in place for **5-8 days⁹** acting as a **microbial barrier⁴**.
- **Reduced risk of needle stick injury¹⁰**.

INDERMIL® flexifuze™



Step 1

Hold the applicator in one hand with the tip pointed upward and away from user and the patient to avoid inadvertent contact.


Step 2

Approximate the wound edges together. Press the device between forefinger and thumb on the dots to break the inner glass ampoule.

Step 3

Gently squeeze the tube between thumb and forefinger until the adhesive appears at the brush tip. Apply adhesive in a thin film over the length of the wound. Hold the wound closed for 60 seconds to complete the bond.

One device provides wound coverage of over 20cm¹¹.

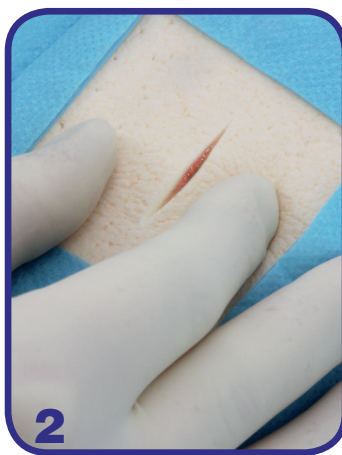

INDERMIL® flexifuze™ Ordering Information		
Item No	Description	Packaging
CM001	INDERMIL® flexifuze™ Topical Tissue Adhesive	x12 - Ampoules per Carton

For further information please contact a Surgical Specialties Representative

Tel: 1300 665 884

Email: service@surgicalspecialties.com.au

References:

1, 2, 4, 5, 7, 8, 9, 11 – Reports on file at Connexicon Medical Ireland.

3 - Swan, MC, Descamps, MJL, and Broadhurst, A. Scar Tattooing Following the Use of Tissue Adhesive Plast. Recon. Surg. 2006; March: 1054-1055

6 - Patent US8047735 B2, EP 1994886 B1 Applicator tip for applying surgical glue. 10 - Gordon CA (2001). Reducing needle stick injuries with the use of 2-octyl cyanoacrylate for laceration repair. American Academy of Nurse Practice.

connexicon
medical

INDERMIL® is a registered trademark of Connexicon Medical Ltd.
flexifuze™ is a trademark of Connexicon Medical Ltd.
© 2018 Connexicon Medical.
Made in Ireland.

[FF-BROC-DIS-Rev005-180605]



**surgical
specialties**

A ParagonCare Brand

1/17 Rodborough Road, Frenchs Forest NSW 2086
PO Box 6052, Frenchs Forest DC NSW 1640, Australia
Customer Service T: 1300 665 884 F: 1300 665 886
www.surgicalspecialties.com.au

4B Ride Way, PO Box 301-218
Albany, Auckland, New Zealand
Customer Service T: 0800 665 884 F: 09 447 1685
www.surgicalspecialties.co.nz